

Coronet Recipient Vacuum Trephine Instructions

- 1) Remove trephine from the packaging
- 2) Align trephine blade with the inner vacuum tube (use the operating microscope)
- 3) **Retract the blade 8 quarter turns**
- 4) Evacuate the syringe
- 5) Place the trephine on the recipient cornea and establish the vacuum
- 6) **Advance the blade 6 quarter turns**
- 7) The blade is now resting on the cornea - Advance the blade carefully, one quarter turn at a time until aqueous is observed.
- 8) Release the vacuum and remove the trephine

