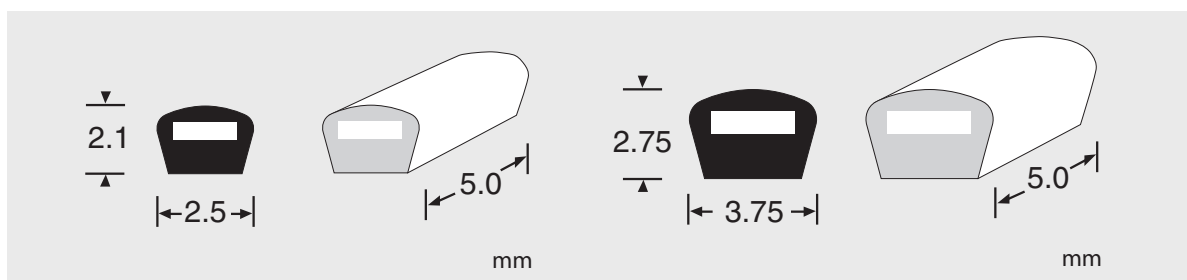
An Addition to the growing “New Generation” of minimal invasive Retinal Implants

- Used to create additional scleral indentation when placed directly over a break.
- Especially useful for small horseshoe shaped tears that “Fishmouth”.
- May be used in conjunction with a vitrectomy when broad support is needed and removal of all traction is not desired.
- Fewer components needed on selected cases.
- Designed to enhance the buckling effect of a sleeve when used in a simple encircling procedure.
- Designed for use with Labtician Circling Bands and Labtician Silicone Strips.
- See reverse for surgical details.

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The Wong Sleeve was developed in cooperation with Dr. David T. Wong, Assistant Professor University of Toronto faculty of Medicine, Toronto, Ontario, Canada

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
ISO 13485

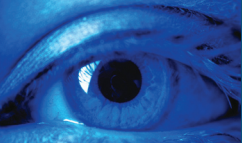ISO 13485 FM72835

Using the Buckling Sleeve as a Buckling Element

by David T. Wong, MD, FRCSC

Introduction

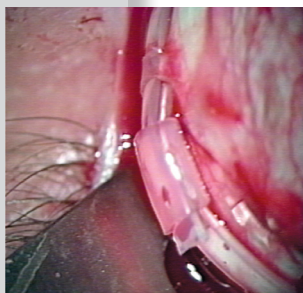
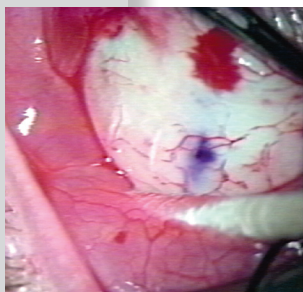
The development of the silicone sleeve has made joining solid silicone bands easier. When using a tire element in conjunction with an encircling band, the silicone sleeve is typically placed 180 degrees from the tire element to reduce bulk and conjunctival erosion.

With the development of minimal buckling techniques where the band becomes the buckling element, the sleeve itself can be used to create additional scleral indentation to further support retinal breaks and reduce vitreous traction.

Technique

A scleral belt loop (or suture loop) is created adjacent to the marked retinal break. The sleeve is placed over the marked break to join the 2 ends of the band. After the band is tightened and the desired indentation is achieved, the sleeve can be further adjusted into position. The effect is similar to a low indentation meridional implant.

Further indentation under the sleeve may be created by placing the belt loop over the retinal break. Position the sleeve within the belt loop, in a manner similar to an intrascleral implant. The belt loop must be sufficiently wide and at least 50% of the scleral thickness to distribute the increased forces.

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
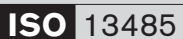
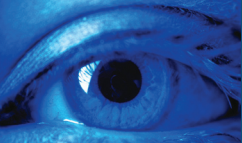
Wide Silicone Strips for Scleral Buckling

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The Labtician Wide Strips offer adequate, but not excessive, width of vitreous base support equally in all quadrants. These strips are designed for use in conjunction with vitrectomy when vitrectomy is used as the primary treatment of retinal breaks. They are particularly useful in cases when there are inferior or large retinal defects and additional prophylaxis is needed. Labtician Wide strips are made from a soft silicone formulation to help prevent erosion.

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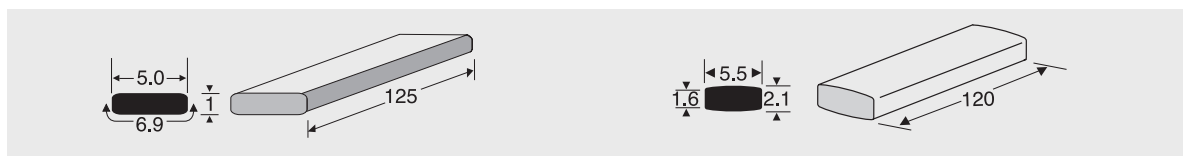S 4050

Width: 5.0 mm
Height: 1.0 mm

Flat Strip can be used with S 3084 Oval Sleeves

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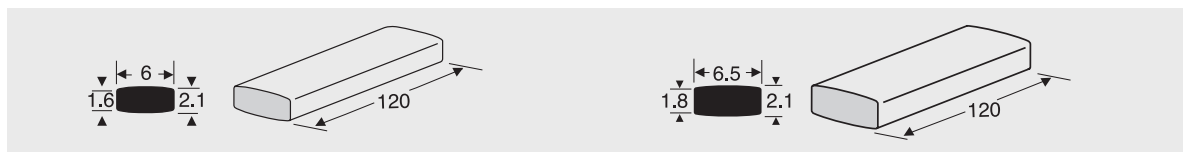
S 4055

Width: 5.5 mm
Height: 1.6 mm shoulders increases to 2.1 mm at centre
Bioconvex shape creates a higher buckle than a flat strip using only one element

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S 4060

Width: 6.0 mm
Height: 1.6 mm shoulders increases to 2.1 mm at centre
Bioconvex shape creates a higher buckle than a flat strip using only one element

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S 4065

Width: 6.5 mm
Height: 1.8 mm shoulders increases to 2.1 mm at centre
Bioconvex shape creates a higher buckle than a flat strip using only one element

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
PARTICIPANT OF THE
CODE OF CONDUCT

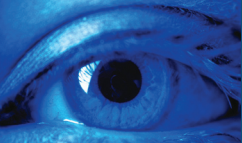
ISO 13485

Dr. David Wong of Toronto, Canada designed both a new Forceps to aid in the placement of encircling elements as well as our unique Oval Sleeve. The Wong Forceps allows easy visualization, grasps encircling elements firmly without damage, and makes it easier to manipulate sutures. Labtician Oval Sleeves greatly simplify and ease the use of sleeves to secure encircling elements. The oval profile is anatomically correct which avoids any puckering effect on the band associated with cylindrical sleeves. Labtician Oval Sleeves are precut to length and are easily placed on the spreader.

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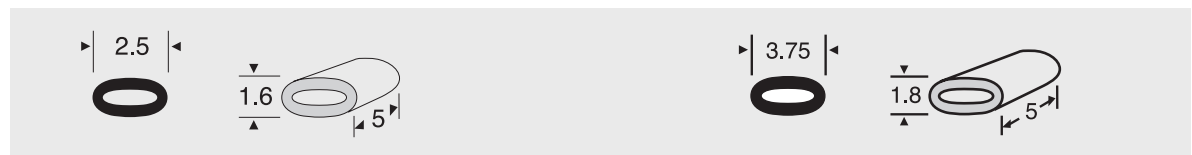S 3083

Fits all Crclng Bands (2.0, 2.5 mm and the Silicone Lace)
Anatomically correct. Precut to length
Easy to place on the Frederick Sleeve Spreader
Buckle can be adjusted at the end of the case

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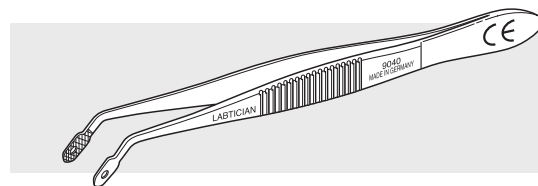
S 3084

Fits all wide flat Silicone Strips (3.5, 4.0, and 5.0 mm)
Anatomically correct. Precut to length
Easy to place on the Frederick Sleeve Spreader
Buckle can be adjusted at the end of the case

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
Frederick Sleeve Spreader
Pince de Frederick

9040

Large grasping platform
Angled for enhanced visualization
Long tips to pass under muscles
Textured grasping surfaces
Unique suture hole to pass sutures under the muscle

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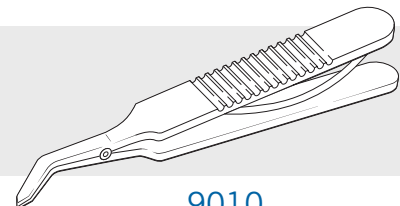
9010

Easy to load and open
Easy to hold open and manoeuver
Produces a sleeve opening much larger than traditional spreaders

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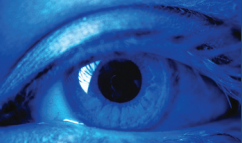
Dr. Scholda designed his new silicone sponges with a dual radius profile that captures the best attributes of round partial thickness sponges in a single design. Studies have shown this new design to give as effective an indentation as round sponges while significantly reducing outer globe contour change. The Scholda design minimizes extrascleral height, reducing the potential rate and degree of irritations, motility disturbances and sponge extrusions.

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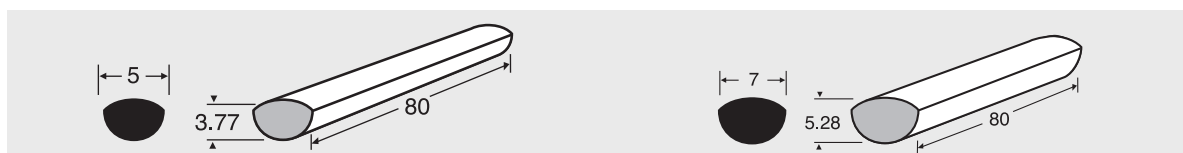S 1985-5

Width: 5.0 mm

- Reduces extraocular bulge
- Good buckle height
- Smooth edges and surfaces
- No sculpting needed
- No exposed open cells contacting sclera
- Consistent shape and size
- Saves time

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Labtician Ophthalmics, Inc.